

Organic Acids Test (OAT)

The Clinical Significance of the Organic Acids Test



The Organic Acids Test (OAT)

Provides an accurate metabolic snapshot of what is going on in the body. Besides offering the most complete and accurate evaluation of intestinal yeast and bacteria, it also provides information on important neurotransmitters, nutritional markers, glutathione status, oxalate metabolism, and much more. The test includes 76 urinary metabolite markers that can be very useful for discovering underlying causes of chronic illness.

Patients and physicians report that treating yeast and bacterial abnormalities reduces fatigue, increases alertness and energy, improves sleep, normalizes bowel function, and reduces hyperactivity and abdominal pain.

The OAT Assists in Evaluating:

- Krebs Cycle Abnormalities
- Neurotransmitter Levels
- Nutritional Deficiencies
- Antioxidant Deficiencies
- Yeast and Clostridia Overgrowth
- Fatty Acid Metabolism
- Oxalate Levels
- And More!

The OAT Pairs Well with the Following Tests:

- GPL-TOX Profile
- IgG Food MAP with Candida + Yeast
- MycoTOX Profile

Learn how to better integrate the OAT into your practice, along with our other top tests by attending one of our MosaicEDGE Practitioner Workshops!

Visit www.MosaicDX.com for workshop dates and locations.

The following pages list the 76 metabolite markers of the Organic Acids Test. Included is the name of the metabolic marker, its clinical significance, and usual initial treatment.

	Intestinal Microbial Overgrowth						
	Yeast and Fungal Markers						
Citramalic Acid	Elevated citramalic acid is produced mainly by Saccharomyces species or Propionibacteria overgrowth. High-potency, multi-strain probiotics may help rebalance GI flora.						
5-Hydroxy-methyl-furoic Acid	A metabolite produced by Aspergillus and possibly other fungal species in the GI tract. Prescription or natural antifungals, along with high-potency, multi-strain probiotics, may reduce overgrowth levels.						
3-Oxoglutaric Acid	Indicates a possible yeast overgrowth in the GI tract. High-potency, multi-strain probiotics may help rebalance GI flora.						
Furan-2,5-dicarboxylic Acid	A metabolite produced by Aspergillus and possibly other fungal species in the GI tract. Prescription or natural antifungals, along with high-potency, multi-strain probiotics, may reduce overgrowth levels.						
Furancarbonylglycine	A metabolite produced by Aspergillus and possibly other fungal species in the GI tract. Prescription or natural antifungals, along with high-potency, multi-strain probiotics, may reduce overgrowth.						
Tartaric Acid	Produced by action of Candida hyaluronidase on the intercellular cement, hyaluronic acid. Oxidation of the hyaluronic acid breakdown by white blood cell hypochlorite produces tartaric acid and arabinose. Antifungal treatment and high-potency, multi-strain probiotics may help rebalance GI flora.						
Arabinose	Produced by action of Candida hyaluronidase on the intercellular cement, hyaluronic acid. Oxidation of the hyaluronic acid breakdown by white blood cell hypochlorite produces tartaric acid and arabinose. Antifungal treatment and high-potency, multi-strain probiotics may help rebalance GI flora.						
Carboxycitric Acid	Elevated yeast/fungal metabolites indicate overgrowth in the GI tract. Prescription or natural antifungals, along with high-potency, multi-strain probiotics, may reduce overgrowth.						
Tricarballylic Acid	A chemical byproduct released from fumonisins during passage through the gastrointestinal tract. Fumonisins are fungal toxins produced primarily by F. verticillioides. Elevated levels can be caused by the intake of corn or corn-based food contaminated with fumonisins.						
	Bacterial Markers						
Hippuric Acid	A bacterial product of phenylalanine metabolism. Most hippuric acid in urine is derived from microbial breakdown of chlorogenic acid, a common substance found in beverages and in many fruits and vegetables. Higher levels indicate GI bacterial overgrowth that can be reduced with natural anti-bacterial agents and/or high-potency, multi-strain probiotics.						
2-Hydroxyphenylacetic Acid	Elevated 2-hydroxyphenylacetic acid is associated with intestinal bacteria overgrowth. High-potency, multi-strain probiotics may help rebalance GI flora.						
4-Hydroxybenzoic Acid	A marker for intestinal dysbiosis. Results may also show elevated values as a result of ingestion of foods such as jams and pie fillings containing paraben preservatives. The use of probiotics and the exclusion of paraben-containing foods is the first treatment consideration.						
4-Hydroxyhippuric Acid	A glycine conjugate of 4-hydroxybenzoic acid, a metabolite of paraben preservatives. May be elevated after exposure to paraben antimicrobials in certain foods and cosmetics. Intake of fruits containing polyphenols rich in anthocyanins, flavonols, and hydroxycinnamates may increase this compound in the urine. Avoid exposure to parabens.						
DHPPA (dihydroxyphenylpropionic acid)	DHPPA in urine indicates intake of chlorogenic acid, a common substance in beverages and many fruits and vegetables. Harmless or beneficial bacteria such as Lactobacilli, Bifidobacteria, and E. coli increase the breakdown of chlorogenic acid to DHPPA, so high values are mainly associated with increased amounts of these species in the GI tract.						

	Clostridia Bacterial Markers						
4-Hydroxyphenylacetic Acid	A tyrosine metabolic product of certain Clostridia bacteria. Elevated levels are associated with Clostridia overgrowth, small intestinal bowel overgrowth (SIBO), or small bowel disease. May also indicate celiac disease.						
HPHPA (3-(3-hydroxyphenyl)- 3-hydroxypropionic acid)	An elevated value indicates an overgrowth of certain Clostridia bacteria in the GI tract includin C. sporogenes , C. caloritolerans , and C. botulinum . This metabolite is commonly elevated in autism, psychiatric disorders, and GI disorders. This metabolite inhibits the enzyme dopamine-beta-hydroxylase, leading to a derangement of neurotransmitter balance.						
4-Cresol	An elevated value indicates an overgrowth in the GI tract of C. difficile . This metabolite is commonly elevated in autism, psychiatric disorders, and GI disorders. This metabolite inhibits the enzyme, dopamine-beta-hydroxylase, leading to a derangement of neurotransmitter balance.						
3-Indoleacetic Acid	3-Indoleacetic acid is a tryptophan byproduct of the following Clostridia species: C. strickland C. lituseburense , C. subterminale , and C. putrefaciens . Very high amounts of this metabolite derived from tryptophan may indicate a depletion of tryptophan needed for other physiologic functions.						
	Oxalate Metabolites						
Glyceric Acid	Elevated in genetic hyperoxaluria type II. Normal values of glyceric acid rule out genetic causes of significant elevation of oxalic acid in urine.						
Glycolic Acid	Very elevated in genetic hyperoxaluria type I. Normal values of glycolic acid rule out genetic causes of significant elevation of oxalic acid in urine. This genetic disease can be confirmed DNA testing at the Mayo Clinic. Glycolic Acid is high in fruits and vegetables.						
Oxalic Acid	Elevated oxalic acid may be associated with dysbiosis from Aspergillus, Penicillium, and possibly Candida , or from high doses of vitamin C. If yeast or fungal markers are elevated, antifungal therapy may reduce oxalates. Elevated oxalic acid may also result from anti-freeze (ethylene glycol) poisoning. Oxalic Acid may also be due to genetic mutations or SNPs in the oxalate pathway.						
	Glycolytic Cycle Metabolites						
Lactic Acid	Elevated by a number of nonspecific influences, such as vigorous exercise, bacterial overgro of the GI tract, shock, poor perfusion, B-vitamin deficiency, mitochondrial dysfunction or damage, and anemia, among others. Tiglylglycine is a more specific indicator of mitochono dysfunction or damage. The possibility of an inborn error of metabolism increases when th lactic acid value exceeds 300 mmol/mol creatinine. There are many inborn errors of metabol that are present with elevated lactic acid, including disorders of sugar metabolism, pyruvate dehydrogenase deficiency, and mitochondrial disorders.						
Pyruvic Acid	Elevated by a number of non-specific factors, including vigorous exercise, bacterial overgrowth of the GI tract, shock, poor perfusion, B-vitamin deficiency, mitochondrial dysfunction or dam- age, and anemia, among others. High pyruvic acid indicates the possibility of an inborn error of metabolism when the value exceeds 100 mmol/mol creatinine.						

	Mitochondrial markers - Krebs Cycle Metabolites
Succinic Acid	Most commonly elevated due to a variety of metal and non-metal toxic chemical exposures. An elevated result may indicate a relative deficiency of riboflavin and/or coenzyme Q10. Also produced by bacterial degradation of unabsorbed glutamine supplement. Low levels may indicate the need for leucine/ isoleucine supplementation. Suggest supplementation with riboflavin and/or coenzyme Q10.
Fumaric Acid	Increased urinary fumaric acid may be due to impaired Krebs cycle function, a defect in the enzyme fumarase, or in mitochondrial function. To support mitochondrial function, supplement with coenzyme Q10, nicotinamide adenine dinucleotide (NAD+), L-carnitine and acetyl-L-carnitine, riboflavin, nicotinamide, biotin, and vitamin E.
Malic Acid	Slightly elevated values usually indicate a higher need for nutrients such as niacin and coenzyme Q10. When malic acid is simultaneously elevated with citric, fumaric, and 2-ketoglutaric acids, a mitochondrial energy pathway dysfunction is strongly suggested.
2-Oxoglutaric Acid	Increased values in urine may be due to dietary vitamin deficiencies or the intake of 2-ketoglutaric acid as a supplement. The conversion of 2-oxoglutaric acid to succinyl-CoA requires coenzyme A (derived from pantothenic acid), flavin adenine dinucleotide (FAD) derived from riboflavin, and thiamine.
Aconitic Acid	Aconitase, the enzyme that metabolizes citric and aconitic acids, is dependent upon glutathione. Elevated in mitochondrial disorders (e.g. Complex I and Pierson Syndrome). Elevated aconitic acid may indicate an additional requirement for reduced glutathione.
Citric Acid	Elevations may be due to increased intake of citric acid-containing foods or result from intestinal yeast-producing citric acid, or perhaps inhibiting the human citric acid cycle. Increased citric acid may also indicate depletion of glutathione, which is required for the enzyme, aconitase to metabolize both aconitic and citric acids. If pyroglutamic acid values are low, consider supplements containing glutathione, or n-acetyl-cysteine.
	Mitochondrial Markers – Amino Acid Metabolites
3-Methylglutaric Acid 3-Methylglutaconic	Significant increase is due to a reduced ability to metabolize the amino acid, leucine. This abnormality is found in the genetic disease, methylglutaconic aciduria and in mitochondrial disorders. 3-Methylglutaconic acid may also be elevated. Supplementation with coenzyme Q10, niacin, L-carnitine and acetyl-L-carnitine, riboflavin, nicotinamide, biotin, and vitamin E may be useful.
	Marker of glutaryl CoA dehydrogenase deficiency. This enzyme is involved in the breakdown of lysine, hydroxylysine, and tryptophan, and is associated with the genetic disease, glutaric aciduria type 1

3-Hydroxyglutaric
 article of glutary Cox denydrogenase denciency. This enzyme is involved in the bleaddown of ysine, hydroxylysine, and tryptophan, and is associated with the genetic disease, glutaric aciduria type 1. Elevated 3-hydroxyglutaric acid may occur in combination with glutaric and glutaconic acids. Symptoms of glutaryl CoA dehydrogenase deficiency are varied. Some individuals appear normal, while others display encephalopathy, autism, cerebral palsy, and additional neurological abnormalities. Treatment includes adherence to a diet low in lysine and supplementation with carnitine. Slight elevations may indicate mitochondrial dysfunction.

Neurotransmitter Metabolism

Phenylalanine and Tyrosine Metabolites

HVA and VMA	HVA (homovanillic acid), a dopamine metabolite, and VMA (vanillyImandelic acid), a metabolite of epinephrine and norepinephrine, are often elevated due to stress increasing catecholamine output from the adrenal gland, or from lead toxicity. Elevated HVA may also result from the intake of L DOPA, dopamine, phenylalanine, or tyrosine, or due to Clostridia metabolites.
HVA/VMA Ratio	An elevated ratio is often the result of a decreased conversion of dopamine to norepinephrine by the enzyme, dopamine beta-hydroxylase. Inhibition of this enzyme is commonly caused by Clostridia by-products, including HPHPA, 4-cresol, and 4-hydroxyphenylacetic acid.
DOPAC	DOPAC (dihydroxyphenylacetic acid) is another major metabolite of dopamine. DOPAC levels may be elevated due to inhibition of dopamine beta hydroxylase (DBH) from Clostridia metabolites, the mold metabolite fusaric acid, pharmaceuticals such as disulfiram, food additives like aspartame, or to deficiencies of the DBH enzyme due to copper deficiency, vitamin C deficiency, or malic acid deficiency. Low production of DOPAC can be due to decreased intake or absorption of dopamine's precursor amino acids such as phenylalanine and/or tyrosine or decreased quantities of cofactors needed for biosynthesis of dopamine such as tetrahydrobiopterin and vitamin B6 coenzyme. DOPAC levels may also be low due to treatment with monoamine oxidase (MAO) inhibitors or genetic polymorphisms (SNPs) of MAO or aldehyde dehydrogenase (ALDH) coding for decreased enzyme activity.
HVA/DOPAC Ratio	An increase in the conversion of DOPAC to HVA might be due to excessive supplementation of S-adenosyl methionine (SAMe) and/or supplements such as methyltetrahydrofolate or methylcobalamin that increase endogenous SAMe. A decrease in the conversion of DOPAC to HVA is most commonly caused by a genetic deficiency of catechol-O-methyltransferase (COMT) and/or a nutritional deficiency of S-adenosyl methionine (SAMe).

	Tryptophan Metabolites							
5-Hydroxyindoleacetic Acid	Metabolite of the neurotransmitter, serotonin. Elevated values may result from supplementing tryptophan or 5 hydroxy tryptophan (5-HTP). Low values may indicate inadequate production of the neurotransmitter, serotonin. Often slightly elevated by ingestion of foods high in serotonir such as avocado, banana, tomato, plum, walnut, pineapple, or eggplant. High values are found in carcinoid syndrome.							
Quinolinic Acid	Increased values in urine may be caused by various factors such as chronic inflammation from microbial infections, central nervous system degeneration, excessive tryptophan supplementation, or even exposure to phthalates. Reduce excess quinolinic acid by eliminati tryptophan supplementation and also reduce exposure to infections and environmental pollutants. Brain damage induced by quinolinic acid can be mitigated by supplements containing acetyl L-carnitine, melatonin, B6, turmeric, and garlic.							
Kynurenic Acid (KYNA)	The most common causes of elevated kynurenic acid are the use of tryptophan supplements and the presence of chronic infections. Very high urine values are found in genetic disorders involving kynureninase deficiency.							
	Pyrimidine Metabolites - Folate Metabolism							
Uracil	Because folic acid is involved as a methyl donor in the conversion of uracil to thymine, elevated uracil may indicate a defect in folic acid metabolism. Elevated uracil is found in about 10% of children with autism.							
Thymine	Slightly elevated urinary thymine has no clinical significance. High values are associated with inflammatory diseases and cancer. Elevated pyrimidines and elevated thymine have been reported in dihydropyrimidine dehydrogenase deficiency, a rare genetic disease, which has been associated with seizures and autism.							
	Ketone and Fatty Acid Oxidation							
3-Hydroxybutyric Acid Acetoacetic Acid	Ketones, such as 3-hydroxybutyric and acetoacetic acids, are the end-products of rapid or excessive fatty acid breakdown. Common causes of elevated ketones are prolonged fasting, protein malnutrition, high fat diet, vitamin B12 deficiency, severe GI Candida overgrowth, and pulmonary infections.							
4-Hydroxybutyric Acid	A moderate urinary increase in 4-hydroxybutyric acid may be due to intake of dietary supplements containing 4-hydroxybutyric acid, also known as gamma-hydroxybutyric acid. Very high levels may indicate the genetic disorder involving succinic semialdehyde dehydrogenase deficiency.							
Adipic Acid	Slightly elevated adipic acid may result from excessive ingestion of gelatin or other "junk" food containing adipic acid as an additive. Elevated adipic acid may also indicate an abnormality in fatty acid metabolism. Dietary supplements containing L-carnitine or L-acetyl-carnitine may be beneficial.							
Suberic Acid Sebacic Acid Ethylmalonic Acid Methylsuccinic Acid	Increased urinary products of the omega fatty acid metabolism pathway may be due to carnitine deficiency, fasting, or increased intake of triglycerides from coconut oil, or some infant formulas. Very elevated values may indicate a genetic disorder. Fatty acid oxidation defects are associated with hypoglycemia and lethargy. Regardless of cause, intake of dietary supplements containing L-carnitine or acetyl-L-carnitine may improve clinical symptoms.							

ciency, , disorder.
f vitamin B6. o not d with
e of ic acid,
urinary and genetic iboflavin and about 10%
cept in be converted ge.
aused by cate bit the ed by
r with eine may armful.
due to ligher levels s and may
nt and only kicity. ts of certain tathione or nt activity of n.
acid r when toxic cessive toxic ain genetic B12,
a may result n errors of mino acids.
ake of aspirin ate levels. t).

Amino Acid Metabolites

2-Hydroxyisovaleric Acid 2-Oxoisovaleric Acid 3-Methyl-2-oxovaleric Acid 2-Hydroxyisocaproic Acid 2-Oxoisocaproic Acid	A moderate increase of branched-chain amino acid metabolites in urine may result from lactic acidosis, episodic ketosis, or deficiencies of the vitamins, thiamine or lipoic acid. Elevated 2-hydroxyisocaproic acid in urine has also been linked to short bowel syndrome. A significant increase of branched-chain amino acid metabolites is associated with the genetic disorders, maple syrup urine disease (MSUD) and pyruvate dehydrogenase deficiency. Patients with slight to moderate elevations may use dietary supplements containing thiamine to improve clinical symptoms.
2-Oxo-4-methiolbutyric Acid	Elevated in an inborn error of methionine metabolism. Confirmation of the genetic disorder requires testing of plasma amino acids.
Mandelic Acid	Increased by dietary phenylalanine or phenylalanine supplementation, but also due to exposure to styrene, a toxic environmental compound. Significant elevation is found in the genetic disorder, phenylketonuria (PKU). A plasma phenylalanine test will rule out PKU.
Phenyllactic Acid	A metabolite of phenylalanine. Elevated values indicate increased intake of dietary phenylalanine or the heterozygous carrier status (homozygosity) for the genetic disease, phenylketonuria (PKU). Values observed in clinically diagnosed PKU typically exceed 200 mmol/mol creatinine.
Phenylpyruvic Acid	Moderate elevations may result from intake of phenylalanine, from genetic carrier status for PKU, or from a deficiency in production of biopterin, a cofactor required for phenylalanine metabolism. Very high values are associated with the genetic disease, PKU.
Homogentisic Acid	Homogentisic acid is elevated in the genetic disorder, homogentisic aciduria (alkaptonuria). Slight increases may indicate the heterozygous genetic carrier state of the disease.
4-Hydroxyphenyllactic Acid	Increased values are commonly associated with tyrosinemias, which can result from immature development of enzyme synthesis in infants or genetic deficiencies. Slight increases may be due to increased tyrosine intake, bacterial gut metabolism, short bowel syndrome, or liver disease.
N-Acetylaspartic Acid	Elevated N-acetylaspartic acid is due to the genetic disorder, Carnavan's disease, a potentially fatal disease causing spongy degeneration of the brain.
Malonic Acid	Associated with the genetic disorders, malonyl-CoA decarboxylase deficiency and malonic aciduria with normal malonyl-CoA decarboxylase activity. Slightly elevated values in urine are unlikely to be clinically significant.
	Pyrimidine Metabolites - Folate Metabolism
Phosphoric Acid	Phosphate urinary excretion is directly proportional to dietary intake. Processed foods high in phosphate include: sodas, candy, ice cream, chocolate, mayonnaise, frozen pizza, commercially baked goods, and meats. Excess phosphate is also associated with hyperparathy- roidism, vitamin D-resistant rickets, immobilization following paraplegia or fracture due to bone resorption, vitamin D intoxication, blood lead levels above 1.5 ppm, renal tubular damage, familial hypophosphatemia, and metabolic acidosis. Low urinary phosphate is most common in low phosphate intake and in vitamin D deficiency.

Requisition #: Patient Name: Date of Birth: Patient Sex:	9900001						
ationt Sex:	Sample Report 12/01/2021 F	Patient Age:		1	Practitioner: Date of Collection: Time of Collection: Report Date:	NO PHYSICIAN 12/01/2022 Not Given 08/04/2023	
*	Organ	ic Acids	Test	- Nutr	itional and Me	tabolic Profile	
Metabolic Markers in Ur	(mmol/m	nce Range ol creatinine)	Patient Value	Reference	e Population - Females U	nder Age 13
Intestinal Microbial	Overgrowth						
Yeast and Fungal Markers 1 Citramalic		5	6.3	1.0			
2 5-Hydroxymethyl-2-fu (Aspergillus)	roic	≤	30	4.0	4.0		
3 3-Oxoglutaric			0.62	H 2.0			2.0
4 Furan-2,6-dicarboxylie (Aspergillus)			22	0	0.00		
5 Furancarbonylglycine (Aspergillus)		≤	3.6	0	0.00 -		
6 Tartaric (Aspergillus)		≤	3.9	H 111			111
7 Arabinose		S	56	0	0.00-		
8 Carboxycitric		≤	34	0	0.00		
9 Tricarballylic (Fusarium)		S	0.86	H 5.0			5.0
Bacterial Markers							
10 Hippuric		5	717	1.0	1.0		
11 2-Hydroxyphenylaceti	0	≤	1.1	H 4.0			4.0
12 4-Hydroxybenzoic		0.09 -	2.0	L 0	0.00		
13 4-Hydroxyhippurio		≤	27	0	0.00		
14 DHPPA (Beneficial Ba	oteria)	≤	0.73	H 7.0			7.0
Clostridia Bacterial Marker	0	<	30	2.0			
(C. difficile, C. stricklandii, C. 16 HPHPA	lituseburense & other	*)	227	5.0			
(C. sporogenes, C. caloritoler 17 4-Cresol	ans, C. botulinum & o	thers)	76	5.0			
(C. difficile) 18 3-Indoleacetic			11	5.0	- 00	<5.0	
15 4-Hydroxyphenylaceti (C. difficile, C. stricklandii, C. 16 HPHPA (C. sporogenes, C. caloritoler 17 4-Cresol	o lituseburense & other	a) ≤ thera)			-20 -50 -50-		

Organis Asids Test - Notritional and Metabolic Peelle Page 1 of 12
Angela Purvis, PhD, NRCC | 9221 Quivira Road, Overland Park, KS 66215 | (913) 341-8949 | Fax: (913) 341-6207 | MosaicOX.com

quisition #: 9900001 tient Name: Sample Report				Practitioner: Date of Collection:	NO PHYSICIAN 12/01/2022	
	ference Rang	10	Patient		Population - Females	Under Ann 42
(mm	ol/mol creatini		Value	Reference	Population - Pennales	Under Age 13
Oxalate Metabolites						
9 Glyceric	0.71	- 9.6	1.0	{10		
0 Glycolic	20	- 202	L 1.0	1.0		
1 Oxalic	16	- 174	L 6.0	5.0		_
Glycolytic Cycle Metabolites						
2 Lactic	0.18	- 44	10	10		
3 Pyruvic	0.88	- 9.1	LO	0.00		
Mitochondrial Markers - Krebs 0	Cycle Metab	olites		<u> </u>		_
4 Succinio		≤ 16	0.50	0.50		_
5 Fumario	0.04	- 1.3	0.50		0.50	
6 Malic 7 2-Oxoglutaric		≤ 2.2 ≤ 81	1.0		<1.0	
7 2-Oxoglutaric 8 Aconitic	11	≤ 81 - 35	1.0 L 1.0	(1.0)		_
9 Citrie	59	- 30	L 1.0	1.0		_
Mitochondrial Markers - Amino			L 1.0	1.0		
mitochonunar markers - Amino	Acia metab	ontes				
0 3-Methylglutaric	0.07	- 0.95	H 1.0	10		
1 3-Hydroxyglutaric		≤ 11	5.0		(5.0	
2 3-Methylglutaconic		≤ 6.4	0	0.00		
Neurotransmitter Metabolites						
nenylalanine and Tyrosine Metabolites 3 Homovanillic (HVA) opamine)		≤ 14	0	0.00		
4 Vanillylmandelic (VMA) orepinephrine, epinephrine)	0.87	- 6.9	1.0	(10		
5 HVA / VMA Ratio	0.12	- 3.0	L O	0.00		
6 Dihydroxyphenylacetic (DOPAC) opamine)	0.07	- 4.0	H 6.0		0	
7 HVA/ DOPAC Ratio	1.6	- 2.6	LO	0.00		_
yptophan Metabolites 8 5-Hydroxyindoleacetic (5-HIAA) erotonin)		≤ 7.7	1.0			_
9 Quinolinic	0.63	- 6.7	LO	0.00		_
0 Kynurenic		≤ 4.1	1.0			

Requisition #: 9900001 Patient Name: Sample Report				Practitioner: Date of Collection:	NO PHYSICIAN 12/01/2022	
(mmo	erence Range I/mol creatinine)		Patient Value	Reference	Population - Females I	Jnder Age 13
Pyrimidine Metabolites - Folate M	letabolism					
41 Uracil	≤	19	0	0.00		
42 Thymine	0.01 -	0.89	L O	0.00		
Ketone and Fatty Acid Oxidation						
				<u> </u>		
43 3-Hydroxybutyric		4.1	0	0.00		
44 Acetoacetic	≤		1.0			
45 Ethylmalonic		4.6	0.60			
46 Methylsuccinic		4.3	0	0.00		_
47 Adipic		9.7	0	0.00		
48 Suberic	s	9.6	0	0.00-		
49 Sebacic	s	0.37	H 1.0			1.0
Nutritional Markers						
/itamin B12 50 Methylmalonic ●	\$	6.2	6.0			50
/itamin B6	-					
51 Pyridoxic (B6)	≤	69	1.0	1.0		
/itamin B6						
62 Pantothenic (B6)	5	26	1.0	1.0		
/itamin B2 (Riboflavin) 53 Glutario ●	s	1.1	1.0			1.0
Vitamin C						
54 Ascorbic	10 -	200	L O	0.00		
Vitamin Q10 (CoQ10)				<u></u>		
55 3-Hydroxy-3-methylglutaric 🟶		101	0	0.00		
Slutathione Precursor and Chelating Agen 56 N-Acetylcysteine (NAC)		0.41	0	0.00		
Biotin (Vitamin H)						
67 Methylcitric *	s	6.6	1.0	1.0		
A high value for this marker may indic	cate a deficiency	of this	s vitamin.			

letabolic Markers in Urine	Reference Range (mmol/mol creatinine)	Patient Value	Reference Population - Females Under Age 13
Indicators of Detoxification	n		
Slutathione 58 Pyroglutamic 🕈	7.0 - 63	5 L 0	0.00
Nethylation, Toxic exposure 59 2-Hydroxybutyric **	≤ 2.	2 H 5.0	60
ammonia Excess 50 Orotio	≤ 0.	88 0	400
Aspartame, salicylates, or GI back 61 2-Hydroxyhippuric	teria ≤ 1.	2 1.0	
 A high value for this marker High values may indicate n Amino Acid Metabolites 			
62 2-Hydroxylsovaleric	≤ 2.	0 1.0	
63 2-Oxoisovalerio	≤ 2.	4 0	0.00
54 3-Methyl-2-oxovaleric	≤ 2.	0 H 3.0	30
55 2-Hydroxyisocaproic	≤ 2.	0 0	0.00
6 2-Oxoisocaproic	≤ 2.	0 0	0.00
57 2-Oxo-4-methiolbutyric	≤ 2.	0 0	0.00
88 Mandelic	≤ 2.	0 1.0	(10)
59 Phenyllactic	≤ 2.	3 0	0.00
70 Phenylpyruvic	≤ 2.	3 0	0.00
71 Homogentisic	≤ 2.	0 0	0.00
72 4-Hydroxyphenyllactic	≤ 2.	0 1.0	(10)
73 N-Acetylaspartic	≤ 38	3 0	0.00-
74 Malonic	≤ 12	2 1.0	10
76 4-Hydroxybutyric	≤ 3.	4 2.3	- 23-
Mineral Metabolism			
76 Phosphoric	1.000 - 7.	300 L 380	380

Practitioner: Date of Collection: NO PHYSICIAN 12/01/2022

 Organic Acids Teter - Nutritional and Metabolis Profile
 Page 3 cd 12
 Organic Acids Teter - Nutritional and Metabolis Profile
 Page 3 cd 12

 Angela Purvis, PhD, NRCC | 9221 Quivira Road, Overland Park, KS 66215 | (913) 341-8949 | Fax: (913) 341-6207 | MosaicDX.com
 Angela Purvis, PhD, NRCC | 9221 Quivira Road, Overland Park, KS 66215 | (913) 341-6207 | MosaicDX.com



All trademarks are owned by the company and its affiliates. ©2023 Mosaic Diagnostics. All Rights Reserved. MDX-OAT-CLINSIG 10-2023 v.1

Visit MosaicDX.com for more resources

(800) 288-0383 customerservice@mosaicdx.com 8400 W 110th Street, Suite 500, Overland Park, KS 66210

If in ♥ ▶

Mosaic Diagnostics
Requisition #: 99000
Patient Name: Sample

9900001 Sample Report