



MOSAIC™
DIAGNOSTICS
Formerly Great Plains Laboratory

Pediatric Collection Kit Instructions

Applying the Pediatric Collection Bag

1. Wash your hands prior to beginning.
2. Cleanse the infant's genital area. Allow the newly cleansed area to air-dry.
3. Peel off backing paper on the collection bag, exposing hypoallergenic adhesive.
4. Separate the infant's legs, then firmly attach the pediatric urine collection bag to the child's genital area.
 - For females: Start from the bottom up with the adhesion at skin flap, separating vagina lips (labia) and work upward. Press firmly to skin surrounding the vagina (labia) with excess of bag toward the feet.



- For males: Position penis naturally through bag opening with excess of bag toward the feet. Press adhesive firmly to entire perimeter without puckering.
5. Re-apply diaper directly over bag. We suggest that you cut a hole in the diaper and pull the pediatric collection bag through which will hold the bag securely in place. If necessary, you may place a second diaper on the outside of the first to prevent the bag from irritating the skin.

After Sample Collection

1. Once the infant urinates into the collection bag, transfer the urine sample from the bag into the specimen collection cup.
2. Refer to Urine Collection Instructions for additional details. For any questions, contact our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383.