



Mold IgE

ALLERGY TEST



Test Details



Sample Type: Blood Draw – Serum

Mold allergy arises when the immune system encounters mold spores or components, usually through inhalation, ingestion, or skin contact. This panel reveals potential triggers for mold allergy symptoms in individuals.

Key Details

- Sample collection must be performed by a phlebotomist.
- Talk to your healthcare provider if you are taking blood-thinning medications or have a blood clotting disorder.
- **Hydrate for 24-48 hours** prior to your blood draw.
- The phlebotomist will collect, prepare, and ship your sample for you.

Scan QR code for
step-by-step
instructions



Before you Start

Use the QR code inside your kit or go to MosaicDX.com/register to register your test. No QR code in your kit? Use paper requisition included.

Collection Preparation

- Check expiration dates on your vials and notify your practitioner if any have expired.
- Have your complete kit, including the full instruction book, ready for your phlebotomist appointment.
- Review included shipping instructions.
- **Complete the barcode sheet** from your test kit, affixing a barcode sticker to each collection sample.
 - Verify that the information is legible and includes the collection date and a matching name for the test order.
 - **Samples lacking labels will be rejected.**

Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



Phone | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



Email | CustomerService@MosaicDX.com



MOSAIC[™]
DIAGNOSTICS
Formerly Great Plains Laboratory

All trademarks are owned by the company and its affiliates.
©2023 Mosaic Diagnostics. All Rights Reserved. MDX-INST-SHEET-IGG-DBS 10-2023 v.1

Visit [MosaicDX.com](https://www.mosaicdx.com) for more resources

(800) 288-0383 customerservice@mosaicdx.com
8400 W 110th Street, Suite 500, Overland Park, KS 66210

