

## Test Details

**Sample Type:** Urine

**Collection Location:** At Home



The MycoTOX Profile provides a comprehensive evaluation of the most prevalent and harmful mycotoxins (toxins produced by mold).

### Key Details

- Do not collect your sample while menstruating.
- You will collect your sample **upon waking, before eating or drinking.**
- **Freeze your sample** before shipping.
- Ship your sample Monday-Thursday.  
For **international shipments**, ship Monday-Wednesday.

Scan QR code for  
step-by-step  
instructions



### Before you Start

Use the QR code inside your kit or go to [MosaicDX.com/register](https://MosaicDX.com/register) to register your test.  
No QR code in your kit? Use paper requisition included.

### Collection Preparation

- **If you are taking mycophenolate mofetil (CellCept/Mycofortic)**, check the appropriate box on the requisition form.
- Fasting for 12 hours before sample collection may enhance mycotoxin detection and test sensitivity
- **Complete the barcode sheet from your test kit, affixing a barcode sticker to each sample.**
  - Verify that the information is legible and includes the collection date and a matching name for the test order.
  - **Samples lacking labels will be rejected.**
- Avoid taking the following at least three days prior to collection:
  - Do not discontinue any medications or supplements without first consulting your healthcare provider.



Glutathione



Charcoal



Bentonite Clay or similar binders

- Review included shipping instructions.

## Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



**Phone** | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



**Email** | [CustomerService@MosaicDX.com](mailto:CustomerService@MosaicDX.com)



**MOSAIC<sup>™</sup>**  
**DIAGNOSTICS**  
*Formerly Great Plains Laboratory*

All trademarks are owned by the company and its affiliates.  
©2023 Mosaic Diagnostics. All Rights Reserved. MDX-INST-SHEET-MYCO 10-2023 v.1

**Visit [MosaicDX.com](https://mosaicdx.com) for more resources**

(800) 288-0383 [customerservice@mosaicdx.com](mailto:customerservice@mosaicdx.com)  
8400 W 110th Street, Suite 500, Overland Park, KS 66210

