



MOSAIC™
DIAGNOSTICS
Formerly Great Plains Laboratory

Shipping Information Within the United States

- **Your sample(s) should already be packaged** into either the Test Kit Box or the Purple Bubble Mailer. If not, please refer to your collection instructions.
- **The provided shipping materials are prepaid,** and no additional fees will need to be provided to UPS. If UPS attempts to bill you for shipping costs, please contact our staff for assistance at 800-288-0383. Mosaic Diagnostics is not responsible for any payments made directly to UPS.


As a friendly reminder, test kits should only be shipped Monday-Thursday except for the stool collections, which should be shipped no later than Wednesday.



- **For Collections in the Test Kit Box:**



Place Test Kit Box into the Laboratory Shipping Pak and seal. You are now ready to ship!

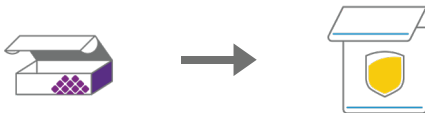

- **For Collections in the Purple Bubble Mailer:**



Your shipping label is attached, and you are ready to ship!

- **Take packaged sample(s) to a UPS Store.** Do not use a UPS dropbox. Visit www.ups.com/dropoff to help find a location near you. You may also request a UPS pick-up by calling 800-742-5877. Let them know that you need to schedule a pick-up using a return label.
- **Take note of the tracking number for your assurance.**