

Test Details

Sample Type: Urine

Collection Location: At Home



The MycoTOX Profile® provides a comprehensive evaluation of the most prevalent and harmful mycotoxins (toxins produced by mold).

Key Details

- Do not collect your sample while menstruating.
- You will collect your sample **upon waking, before eating or drinking.**
- **Freeze your sample** before shipping.
- Ship your sample Monday-Thursday.
For **international shipments**, ship Monday-Wednesday.

Scan QR code for
step-by-step
instructions



Collection Preparation

- **If you are taking mycophenolate mofetil (CellCept/Mycofortic)**, check the appropriate box on the requisition form.
- Fasting for 12 hours before sample collection may enhance mycotoxin detection and test sensitivity
- **Complete the barcode sheet from your test kit, affixing a barcode sticker to each sample.**
 - Verify that the information is legible and includes the collection date and a matching name for the test order.
 - **Samples lacking a complete name and labels will be rejected.**
- Avoid taking the following at least three days prior to collection:
 - Do not discontinue any medications or supplements without first consulting your healthcare provider.



Glutathione



Charcoal



Bentonite Clay or similar binders

- Review included shipping instructions.

Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



Phone | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



Email | CustomerService@MosaicDX.com



MOSAIC[™]
DIAGNOSTICS
Formerly Great Plains Laboratory

All trademarks are owned by the company and its affiliates.
©2025 Mosaic Diagnostics. All Rights Reserved. MDX-INST-SHEET-MYCO 1-2025 v.3

Visit [MosaicDX.com](https://mosaicdx.com) for more resources

(800) 288-0383 customerservice@mosaicdx.com
8400 W 110th Street, Suite 500, Overland Park, KS 66210

