



TOXDetect
PROFILE

Test Details

Sample Type: Urine

Collection Location: At Home



The TOXDetect Profile provides a comprehensive assessment of exposure to environmental toxicants.

Key Details

- Do not collect your sample while menstruating.
- You will collect your sample **upon waking, before eating or drinking.**
- **Freeze your sample** before shipping.
- Ship your sample Monday-Thursday.
For **international shipments**, ship Monday-Wednesday.

Scan QR code for
step-by-step
instructions



Collection Preparation

- If you are taking the antibiotic Cephalexin (Cefalexin, Keflex, Bio-Cef, Novo-Lexin), please wait a minimum of four days after the last dose to collect your urine sample. Please consult your practitioner for your personalized care plan.
- Place your ice pack flat in your freezer, so it will be ready to ship with your specimen.
- Review included shipping instructions.
- **Complete the barcode sheet** from your test kit, affixing a barcode sticker to each sample
 - Verify that the information is legible and includes the collection date and a matching name for the test order.
 - **Samples lacking a complete name and labels will be rejected.**

Any Questions?

If you have questions about any aspect of the specimen collection or shipping process, please feel free to contact us:



Phone | our friendly customer service team is available Monday through Friday 8am-5pm CST at 800-288-0383



Email | CustomerService@MosaicDX.com



MOSAIC[™]
DIAGNOSTICS
Formerly Great Plains Laboratory

All trademarks are owned by the company and its affiliates.
©2023 Mosaic Diagnostics. All Rights Reserved. MDX-INST-SHEET-TOX 11-2024 v.2

Visit [MosaicDX.com](https://www.mosaicdx.com) for more resources

(800) 288-0383 customerservice@mosaicdx.com
8400 W 110th Street, Suite 500, Overland Park, KS 66210

