

# The **DUST** Test

## **STEP 1** REGISTER

Register your kit with the QR code on the inside flap. Your assigned Home Care Specialist will be notified and will be your point of contact throughout the process.

**UNREGISTERED SAMPLES WILL BE DISCARDED.**

## **STEP 2** COLLECT

Collect your dust sample with the included materials. Full instructions are located on the back of this card.

## **STEP 3** SHIP

Place your dust sample in the prepaid envelope and Drop it off at the carrier. Track your shipment in your online portal.

## **STEP 4** RESULTS

Your results will be available in your online portal, where you can check the status at any time. Standard processing is up to approx. 1 week from the time your sample reaches the lab, unless an expedited process was purchased.

## **STEP 5** BOOK YOUR CALL

Book a call with your Home Care Specialist to review your results and discuss the next steps based on your situation.

*Watch a 10 minute video walkthrough here:  
[www.thedusttest.com/tutorial](http://www.thedusttest.com/tutorial)*

# COLLECTION INSTRUCTIONS

1. Put on the plastic gloves to avoid cross-contamination.
2. Remove the collection cloth from its sealed packaging.
3. Wipe up dust with the cloth from surfaces throughout your home.
4. Some substances like rust, drywall dust, and more intensive cleaning chemicals can inhibit laboratory analysis. It is best to avoid including those in the sample, and we recommend avoiding bathrooms and kitchens, and cleaning after any construction or renovation. If inhibitors are detected, the lab will not run the test and there would be the option to resample.
5. Wipe up as much dust as possible, ideally covering the entire collection cloth on both sides. (If you have recently cleaned, wait for more dust to accumulate before testing.)
6. With the collection cloth now covered in dust, place it inside the included ziploc bag and seal the bag closed.
7. Place your ziploc bag into the prepaid mailer envelope and drop it off.



**IMPORTANT: GET  
AS MUCH DUST  
AS POSSIBLE.**

Watch the videos at  
[thedusttest.com/tutorial](https://thedusttest.com/tutorial)  
to see this in action.

**\*If insufficient dust is collected,  
laboratory analysis cannot be  
performed and a new kit will need  
to be purchased for recollection.**