

Test Collection Instructions - DNA Signature Profiles

1. Do not eat, drink, brush your teeth, use mouthwash, oils, oral analgesics, adhesives, tobacco, or nicotine for at least 30 minutes before collection. Rinsing your mouth with water before sampling is okay.
2. Tap the vial gently 2-3 times. Unscrew the white cap and stand the vial upright.
3. Rub the swab firmly along the inside of both cheeks 15-20 times.
4. Insert the swab into the vial using a corkscrew motion until fully submerged.
5. Move the swab up and down rapidly 10-15 times while keeping it in the liquid.
6. Remove the swab with a corkscrew motion and discard it.
7. Replace the cap tightly, place the vial in the biohazard bag, then into the prepaid mailer and seal.
8. Drop the package at your local USPS office. Write down and save the tracking number.



Any Questions? If you have questions about any aspect of the specimen collection or shipping process please contact us Mon-Fri 8AM to 5PM CST at 800-288-0383

Test Collection Instructions - DNA Signature Profiles



Log In to the Patient Portal

Scan QR Code above or visit <https://mosaicdx.com/portal/register/patient>
to create or log in to the Mosaic Diagnostics patient portal.